



**RETRO
PSOAS**


EARP™

Endoscopic Assisted Retropsoas Approach
A Smarter Path to Spinal Reconstruction



**IONM
ALERT PLAN**

IONM Alert & Intervention Response Plan

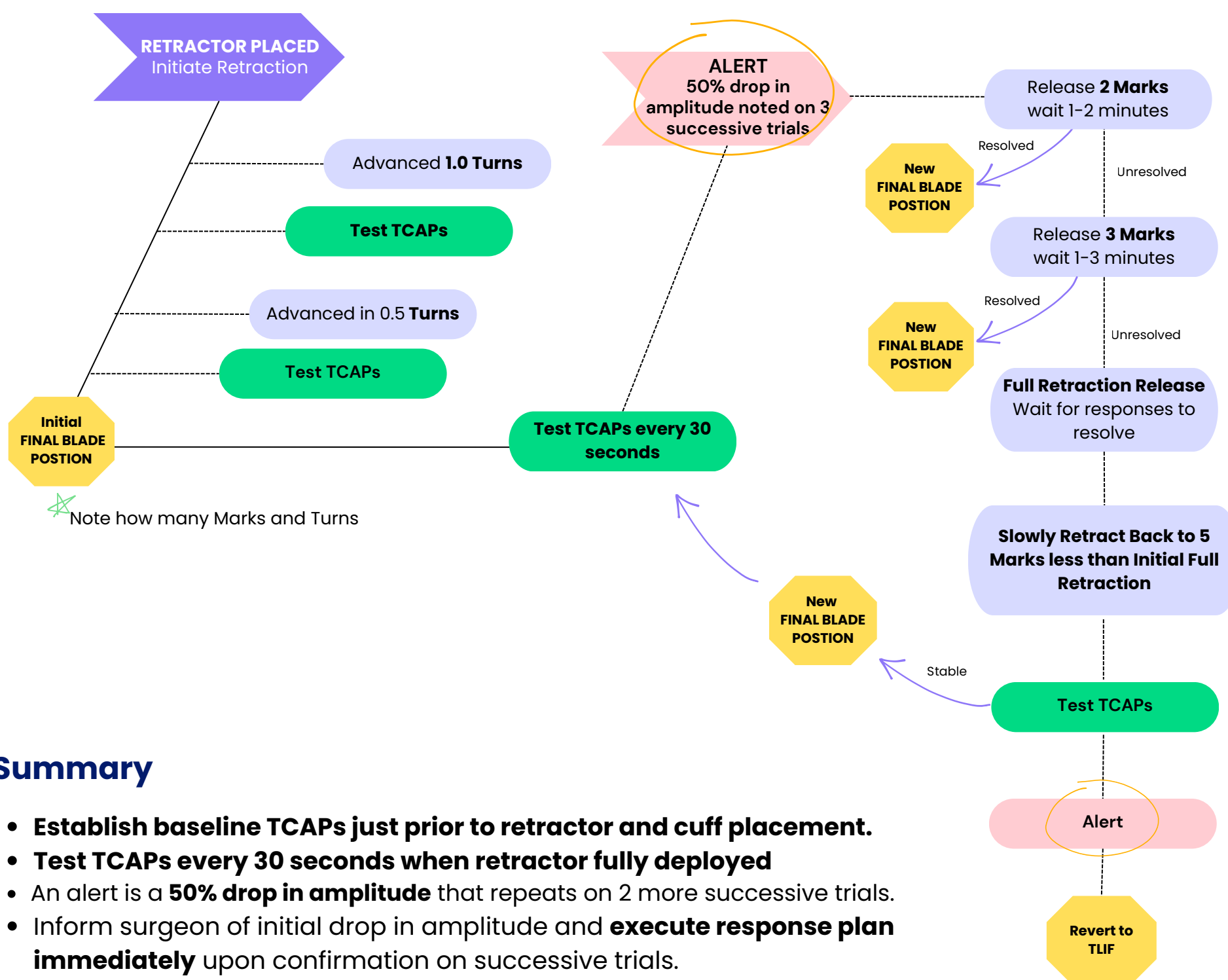

Summary

- Establish baseline TCAPs just prior to retractor and cuff placement.
- Test TCAPs every 30 seconds when retractor fully deployed
- An alert is a **50% drop in amplitude** that repeats on 2 more successive trials.
- Inform surgeon of initial drop in amplitude and **execute response plan immediately** upon confirmation on successive trials.
- Responses are considered resolved if within 20-30% of baseline values

One Position. One Incision. Faster Recovery.

✉ bwilent@retropsoas.com

☎ 724-877-7600

🌐 www.retropsoas.com